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 **Operating instructions:** For thermoforming machines with thermoforming time control.

Enter data (seconds and/or codes) and thermoforming.

For thermoforming machines without control, once the heating process is activated, it is necessary to detect when the sheet gets a concave shape (slightly accentuated) and thermoforming.

**Cool to room temperature** before removing the sheet from the thermoforming machine.

**For an optimal finish, use tools and materials specifically designed for our sheets** as shown in the flyer inside the package.

**After finishing**, spray the aligner with a 90° neutral ethyl solution, lift a flap and remove the protective films, by using a pointed tool.

To avoid any deformations, do not use hot water or steam jet.

Our sheets are compatible with light-curing (Clear Resin code TP-001-309) and acrylic resins.

They are highly biocompatible and they have characteristics of memory, resilience and transparency that make them unique. **For further details regarding the product, please refer to the technical data sheet to be requested to the manufacturing company Orthodontics High Design srl.**

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