# COLD<sup>TM</sup> Collection Smart Pliers

By ThermalOrth Philosophy System

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# COLD<sup>TM</sup> Collection Smart Pliers

By ThermalOrth Philosophy System

*They are the most advanced technological expression of instruments designed to be used on aligners.*

*The Cold Collection Smart Pliers® do not need to be thermoregulated, since they work cold on 0.5 - 0.625 - 0.75 - 0.85 - 1.0mm. aligners. A greater thickness could compromise the functionality of the instrument. With the Cold Pliers you can instantly modify the aligners in the dental practice without interrupting the treatment.*

*They are easy to use and allow multiple solutions on patients who wear aligners.*

## Half Moon


TP-001-084/SM


*It removes the plastic on the aligner with great precision, for the insertion of buttons, accessories, etc*

## Hole Key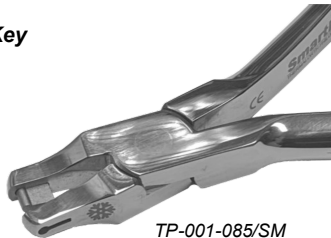

TP-001-085/SM

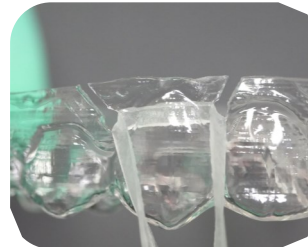
*It makes holes on the aligner for the insertion of elastics for 2nd/3rd classes tractions, intrusions, etc...*

## Triangular


TP-001-087/SM

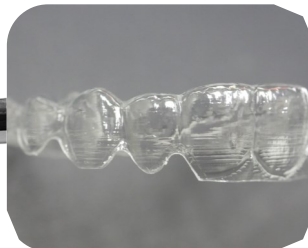
*It perfectly contours the edges of plastics for a very fast and precise aligner finishing.*

## Aligner Punch

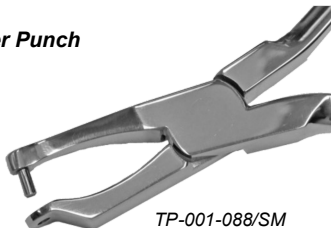TP-001-088/SM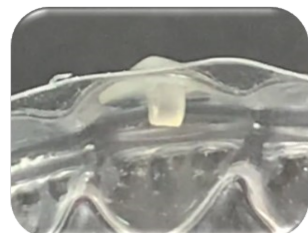

*It makes holes to adhere to attachments by increasing the aligner retention or to insert an elastic for dental movements.*

**Warranty** - The pliers are guaranteed for 5 years. The warranty is void in case of improper use, tampering or use of non-original spare parts. The warranty does not cover labor and travel expenses, which will always and in any case be borne by the buyer.

**Maintenance Instructions** - The Cold Collection Smart pliers are manufactured by using the Japanese medical stainless steel to guarantee a longer duration over time. They must be washed and dried thoroughly. Subsequently they must be lubricated with synthetic oil code 001-115 (only one drop). The main causes of spots and discolorations are due to a lubricant excess and to an incorrect cleaning (washing and drying not properly carried out) before sterilization. Any yellow or brown spots can be removed by rubbing the pliers with extra fine steel wool.

**For further details regarding the product, please refer to the technical data sheet to be requested to the manufacturer Orthodontics High Design srl.**

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